
ARION
Easy-Slide
Kids

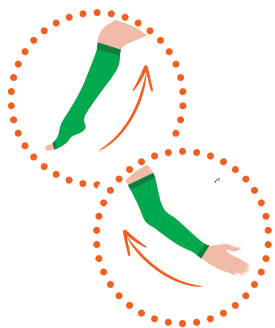Arion Easy-Slide Kids

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
Arion Easy-Slide Kids

Donning aid for open-toe medical compression stockings and for medical compression arm sleeves. For children under 7 (approximately).

Important information before you start using the product:

- A.** Useful tips:
- Wear special non-slip gloves when using the product. This gives you maximum grip on the stocking and reduces the physical strength needed.
 - In case the compression garment has a silicon border, make sure to turn it outwards.
- B.** Make sure the product is ready for use:
- Fold back the extra fabric into the „green bag“. For this, you can use a long spoon if necessary.
 - Align the fold line with the top of the orange label.

Donning instructions:

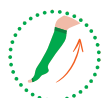1. Slide the arm or leg of the child in the Easy-Slide Kids

2. Slide the arm or leg sleeve over the Easy-Slide Kids


3. Remove the Easy-Slide Kids gently out of the stocking, leg or arm sleeve by pulling the green loop while holding the top of the stocking or sleeve so that it does not slip.

On

SIZING

REF	Size	Length
Eo 1700	Extra small	29 ^{1/2} cm / 11 ^{1/2} "
Eo 1701	Small	39 ^{1/2} cm / 15 ^{1/2} "

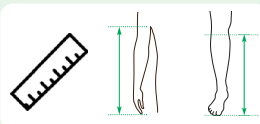Easy-Slide kids from Arion Slide Solutions minimizes friction between compression garment and skin thanks to the product's unique sliding properties provided by the special material it is made of and by its coating. The friction reduction is particularly helpful to slide a stocking or tights over the heel area, which is the broadest part of the foot.

EASY, SAFE AND SIMPLE SOLUTION

- Reduces physical strain for both compression wearers and caregivers
- May contribute to a longer life for the garment and to maintaining the original compression longer by preventing damage from fingers or nails and avoiding overstretching and distortion
- Promotes independence of compression wearers and supports therapy compliance

COMPOSITION

- Product: Polyamide 93%, Polyurethane 3%, Silicone 2%, Polyester 2%
- Package: Low-density polyethylene (LDPE) 

CONTRAINDICATIONS

- Do not use the product if you are allergic to one of its component (see composition).
- Open wounds should be covered with an appropriate dressing before using the product.

⚠️ WARNINGS

- **Suffocation hazard:** Keep the product away from babies, children, and pets.
- **Slipping hazard:** Do not step on the product. Highly slippery material.
- **For external use only /** Intended for skin contact. Seek medical attention if irritation develops.

IMPORTANT

-  Please **read the instructions for use carefully**. If you have any questions, please contact your reseller.
- Please **keep the package and the instructions for use during the time of use of the product**.
- All **serious incidents** that occur when using the product **should be reported** to the manufacturer and the competent supervisory authority.
- When using the product it is important to **use the right product size** for the user. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly. The correct size is determined by your shoe size.
- **Use the product on a single person.** If the product needs to be used on more than one person, disinfect the product before with a 70% ethanol solution.

STORAGE

Keep away from direct sunlight or other heat sources. 

CARE INSTRUCTIONS

- Wash by hand, at max. 30°C/86°F.
- Do not bleach.
- Do not tumble dry. Dry the product only with a towel.
- Do not iron.
- Do not dry clean.
- Disinfection possible with a 70% ethanol solution.

GUARANTEE

This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. The guarantee does not apply if the defect is due to improper use of the product or use on several patients. To use this service you need to contact your supplier. You are asked to return the product washed and to follow the care instructions.

RECOMMENDATION FOR USE

 The manufacturer recommends replacing the product six months after the first use if the product is used regularly.

You can write the date of first use on the back of this document.

Watch our **instruction video** here: uqr.to/ESKidsEN or scan this QR code:

More information on www.slide-solutions.com

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